

RDNZ Customer Bulletin

DATE: May 26 2008

REFERENCE: CB 08/18

Reassignment of Cholesterol Set Points and Target Values ID-MS Standardisation

Current Situation:

A routine Investigation of cholesterol with the Cfas master lot (standardised against ID-MS) has revealed that the set point has to be adjusted by +4%. As a result we have been informed that set points for Cfas and target values for Precinorm U Plus/ Precipath U Plus, Precinorm U/Precipath U, Precinorm L and Precipath L have been reassigned.

Actions Required:

Consult the attached value sheet and edit the set point for the Cfas calibrator and the target value for the appropriate control(s).

We apologise for any inconvenience. Please contact us on Techline if you would like to discuss this further, or require any additional information. .



IMPORTANT NOTICE

**WICHTIGER HINWEIS / REMARQUE IMPORTANTE
 NOTA IMPORTANTE / NOTA IMPORTANTE
 NOTA IMPORTANTE / VIKTIG ANMÄRKNING
 VIGTIGT / DŮLEŽITÉ UPOZORNĚNÍ
 DŮLEŽITÉ UPOZORNENIE / WAŻNE INFORMACJE
 FONTOS MEGJEGYZÉS / ΣΗΜΑΝΤΙΚΗ ΣΗΜΕΙΩΣΗ**

Please refer to the new assigned values:
 Bitte beachten Sie die neu ermittelten Sollwerte:
 Veuillez tenir compte des nouvelles valeurs théoriques:
 Observe los nuevos valores teóricos:
 Si prega di prendere in considerazione i nuovi valori teorici:
 Consulte os novos valores teóricos:
 Vänligen beakta de nya åsatta värdena:
 Se de nye angivne værdier:
 Věnujte pozornost novým určeným hodnotám :
 Prosím použite novo definované hodnoty:
 Należy odnieść się do nowo wyznaczonych wartości:
 Nézze meg az új vonatkozó értékeket:
 Ανατράξτε στις νέες καθορισμένες τιμές:

Cholesterol CHOD-PAP, ID/MS

Roche/Hitachi Systems

03039757 190, **11491458** 216, **11875523** 216,
11875540 216, **11489232** 216, **11489437** 216,
12016630 122

CHOL : ACN 339, 432, 433, 798

Serum / Sérum / Suero / Siero / Soro / Sérum / Surowica / Szérum / Ορός
 Plasma / Plazma / Osocze / Πλάσμα

	mg/dL	mmol/L	g/L
LOT	Calibration Value/Kalibrationswert	Calibration Value/Kalibrationswert	Calibration Value/Kalibrationswert
	Valeur de calibration/Valor de calibración	Valeur de calibration/Valor de calibración	Valeur de calibration/Valor de calibración
	Valore di calibrazione/Valor de calibração	Valore di calibrazione/Valor de calibração	Valore di calibrazione/Valor de calibração
	Kalibreringsværdi/Kalibreringsværdi	Kalibreringsværdi/Kalibreringsværdi	Kalibreringsværdi/Kalibreringsværdi
	Kalibrační hodnota/Hodnota kalibrácie	Kalibrační hodnota/Hodnota kalibrácie	Kalibrační hodnota/Hodnota kalibrácie
	Wartość kalibratora/Kalibrációs érték	Wartość kalibratora/Kalibrációs érték	Wartość kalibratora/Kalibrációs érték
	Τιμή βαθμονόμησης	Τιμή βαθμονόμησης	Τιμή βαθμονόμησης

C.f.a.s., No. 10759350 190

177151	164	4.25	1.64
177152	164	4.25	1.64
177153	164	4.25	1.64
177154	164	4.25	1.64
177463	162	4.20	1.62
177464	163	4.22	1.63
177809	163	4.22	1.63
178815	166	4.30	1.66
178816	165	4.27	1.65
178817	165	4.27	1.65
178818	167	4.33	1.67
178819	166	4.30	1.66
178820	166	4.30	1.66
179799	167	4.33	1.67
179800	167	4.33	1.67

LOT	Roche mg/dL			Roche mmol/L			Roche g/L		
	Value/Wert	Range/Bereich	1SD/1SD	Value/Wert	Range/Bereich	1SD/1SD	Value/Wert	Range/Bereich	1SD/1SD
	Valeur/Valor	Intervalle/Intervalo	1s/1DE	Valeur/Valor	Intervalle/Intervalo	1s/1DE	Valeur/Valor	Intervalle/Intervalo	1s/1DE
	Valore/Valor	Intervallo/Intervalo	1DS/1DP	Valore/Valor	Intervallo/Intervalo	1DS/1DP	Valore/Valor	Intervallo/Intervalo	1DS/1DP
	Värde/Værdi	Område/Område	1DP/1SD	Värde/Værdi	Område/Område	1DP/1SD	Värde/Værdi	Område/Område	1DP/1SD
	Hornota/Hodnota	Rozmezí/Rozsah	1SD/1SD	Hornota/Hodnota	Rozmezí/Rozsah	1SD/1SD	Hornota/Hodnota	Rozmezí/Rozsah	1SD/1SD
	Wartość/Érték	Zakres/Tartomány	1SD/1SD	Wartość/Érték	Zakres/Tartomány	1SD/1SD	Wartość/Érték	Zakres/Tartomány	1SD/1SD
	Τιμή	Εύρος	1SD	Τιμή	Εύρος	1SD	Τιμή	Εύρος	1SD

Precinorm U, No. 10171735 122, 10171743 122, 10651257 922 (Q.C.S.)

173499	97.0	82.3 - 111.7	4.9	2.51	2.12 - 2.90	0.13	0.970	0.823 - 1.117	0.049
173501	97.0	82.3 - 111.7	4.9	2.51	2.12 - 2.90	0.13	0.970	0.823 - 1.117	0.049
175650	84.3	71.7 - 96.9	4.2	2.18	1.85 - 2.51	0.11	0.843	0.717 - 0.969	0.042
176136	93.6	79.5 - 107.7	4.7	2.42	2.06 - 2.78	0.12	0.936	0.795 - 1.077	0.047
176138	93.6	79.5 - 107.7	4.7	2.42	2.06 - 2.78	0.12	0.936	0.795 - 1.077	0.047
176469	94.7	80.6 - 108.8	4.7	2.45	2.09 - 2.81	0.12	0.947	0.806 - 1.088	0.047
176830	96.7	82.3 - 111.1	4.8	2.50	2.11 - 2.89	0.13	0.967	0.823 - 1.111	0.048
176831	96.7	82.3 - 111.1	4.8	2.50	2.11 - 2.89	0.13	0.967	0.823 - 1.111	0.048
178985	92.4	78.6 - 106.2	4.6	2.39	2.03 - 2.75	0.12	0.924	0.786 - 1.062	0.046
178988	92.4	78.6 - 106.2	4.6	2.39	2.03 - 2.75	0.12	0.924	0.786 - 1.062	0.046
179596	102	87 - 117	5	2.64	2.25 - 3.03	0.13	1.02	0.87 - 1.17	0.05
179599	102	87 - 117	5	2.64	2.25 - 3.03	0.13	1.02	0.87 - 1.17	0.05

Precinorm U plus, No. 12149435 122

172222	64.6	55.0 - 74.2	3.2	1.67	1.43 - 1.91	0.08	0.646	0.550 - 0.742	0.032
175487	74.6	63.5 - 85.7	3.7	1.93	1.63 - 2.23	0.10	0.746	0.635 - 0.857	0.037
177326	90.5	77.0 - 104.0	4.5	2.34	1.98 - 2.70	0.12	0.905	0.770 - 1.040	0.045
177328	90.5	77.0 - 104.0	4.5	2.34	1.98 - 2.70	0.12	0.905	0.770 - 1.040	0.045
179445	92.0	78.2 - 105.8	4.6	2.38	2.02 - 2.74	0.12	0.920	0.782 - 1.058	0.046

Precinorm Lipid, No. 10781827 122, 11240161 922 (Q.C.S.)

171746	196	166 - 226	10	5.08	4.33 - 5.83	0.25	1.96	1.66 - 2.26	0.10
174951	189	162 - 216	9	4.90	4.15 - 5.65	0.25	1.89	1.62 - 2.16	0.09
178148	191	161 - 221	10	4.95	4.20 - 5.70	0.25	1.91	1.61 - 2.21	0.10

Precipath U, No. 10171760 122, 10171778 122, 10651265 922 (Q.C.S.)

173043	194	164 - 224	10	5.02	4.27 - 5.77	0.25	1.94	1.64 - 2.24	0.10
174531	201	171 - 231	10	5.21	4.43 - 5.99	0.26	2.01	1.71 - 2.31	0.10
175332	208	178 - 238	10	5.39	4.58 - 6.20	0.27	2.08	1.78 - 2.38	0.10
175333	208	178 - 238	10	5.39	4.58 - 6.20	0.27	2.08	1.78 - 2.38	0.10
176287	205	175 - 235	10	5.31	4.50 - 6.12	0.27	2.05	1.75 - 2.35	0.10
177995	213	180 - 246	11	5.52	4.68 - 6.36	0.28	2.13	1.80 - 2.46	0.11
177996	213	180 - 246	11	5.52	4.68 - 6.36	0.28	2.13	1.80 - 2.46	0.11
179051	206	176 - 236	10	5.34	4.53 - 6.15	0.27	2.06	1.76 - 2.36	0.10

Precipath U plus, No. 12149443 122

176975	192	162 - 222	10	4.97	4.22 - 5.72	0.25	1.92	1.62 - 2.22	0.10
178012	202	172 - 232	10	5.23	4.45 - 6.01	0.26	2.02	1.72 - 2.32	0.10
178014	202	172 - 232	10	5.23	4.45 - 6.01	0.26	2.02	1.72 - 2.32	0.10

Precipath Lipid, No. 11285874 122, 11291114 922 (Q.C.S.)

176680	262	223 - 301	13	6.79	5.77 - 7.81	0.34	2.62	2.23 - 3.01	0.13
179331	290	245 - 335	15	7.51	6.37 - 8.65	0.38	2.90	2.45 - 3.35	0.15