

RDNZ Customer Bulletin

DATE: August 4th 2008

REFERENCE: CB 08/27

Increase Sensitivity to Methamphetamine of Amphetamine Assay

Current Situation:

We have been advised that following a customer report, the Amphetamine assay, lot number 156063, has a greater apparent cross-reactivity to methamphetamine than stated in the package insert.

Actions Required:

Please be aware that materials that contain methamphetamine alone, such as QAP material, are more likely to screen positive with this lot number of the amphetamine assay compared with other lot numbers. Amphetamine detection and confirmation is unaffected.

Please note that only lot number 156063 is affected.

We apologise for any inconvenience. Please contact us on Techline if you would like to discuss this further, or require any additional information. .

Roche Ref: PB 2008/049